

From serial no. 8270.26.001

Drawn by: KA	Date: 19-10-2023
Revised: 1 /KA	Date: 31-07-2024
Revised: 2 /KA	Date: 15-01-2026

# Overview parts Lotus XL

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Draw. no.: EXPV8270-2025-02

## Lotus XL, Article no: 5900 8270

Part no.	Art.no.	QTY.	Name	Drawing no.
1	70827002	1	Chassis complete	8270-Chassis complete
2	70820002	1	Seat frame (set)	8200-12.3
3	70820003	1	Frame handgrip	8270-13
4	70827004	1	Knee support plate	8270-15
5	70820014	4	Leaf spring	8200-36
6	70820004	2	Seat shell	8200-21.5
7	70820005	2	Seat cap	8200-22
8	70820013	2	Cap	8200-23.1
9	70820009	2	Tube cap	8200-27
10	70820010	1	Knee support	8200-30.2
11	70560049	2	Handgrip	5600-33.1
12	70827007	1	Footplate XL	8270-1105
13	70820019	4	Cross Knob PA 40 mm M6 grey	Kruisknop M6
14	70820031	2	Slide bearing 8x15x10	8200-39
15		4	Screw M4x12 DIN7985 A2	M4x8
16		2	Screw M6x10 DIN965 A2	M6x10 DIN965
17		4	Locknut M6 DIN985 A2	M6 DIN985
18		4	Nut cap M6	Moerkap M6
19	70820021	4	Screw M8x16 ISO7380 A2 Dry-loc+Lurinox	M8x16 ISO7380 Dry-loc+Lurinox
20		2	Screw M8x20 A2 ISO7380 flange	M8x20 ISO7380 Kraag



Drawn by: KA	Date: 19-10-2023
Revised: 1 /KA	Date: 31-07-2024
Revised: 2 /KA	Date: 15-01-2026


# Overview parts Lotus XL

Copyright reserved  
by law

Unit of measure:  
mm

Lopital Nederland bv

Draw. no.: EXPV8270-2025-02

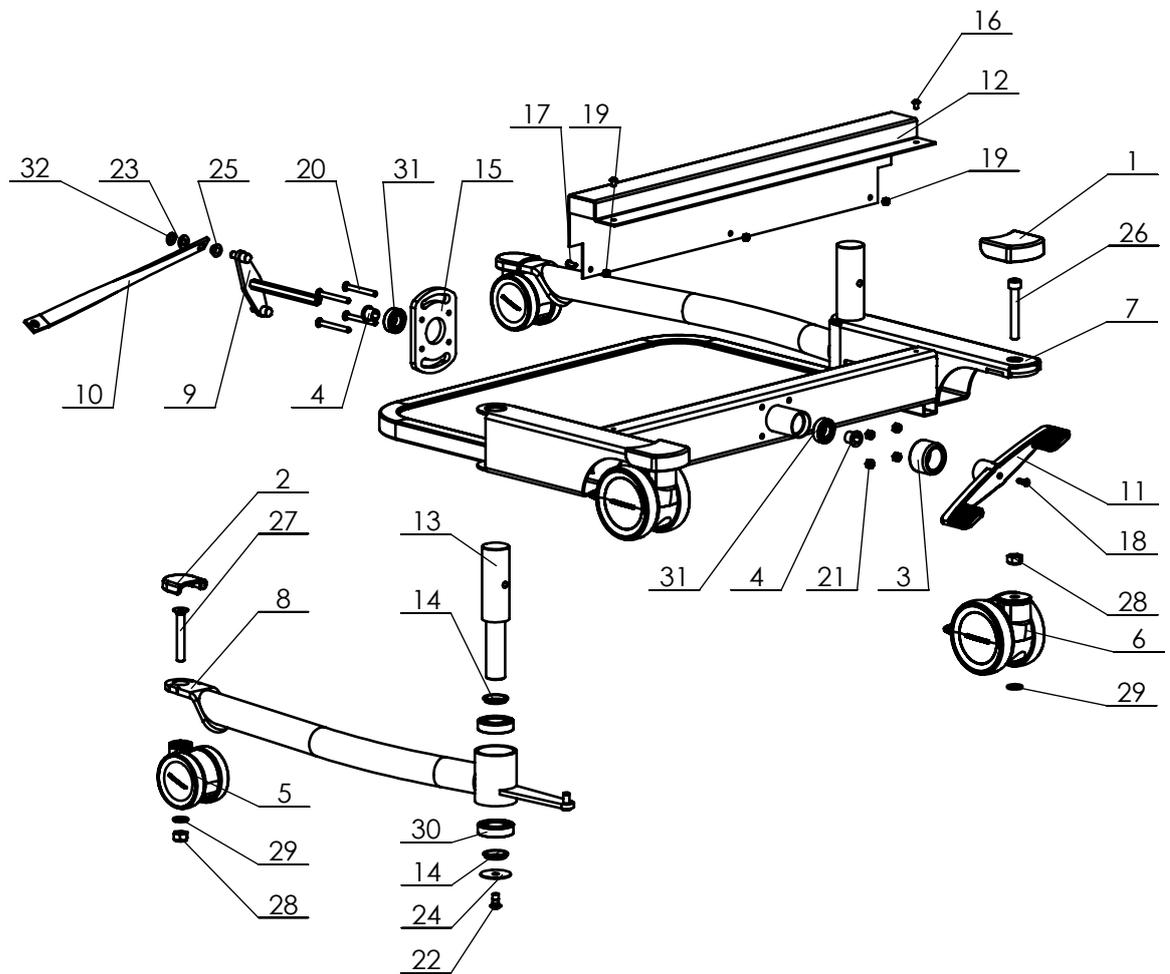

P:\SW-Tek\Lotus 8200\

Drawn by: KA	Date: 19-10-2023
Revised: <b>1</b> /KA	Date: 31-07-2024
Revised: <b>2</b> /KA	Date: 16-01-2026

## Overview parts Chassis complete Lotus XL

Copyright reserved  
by law


Unit of measure-  
ment: mm

Lopital Nederland bv

Draw. no.: EXPV8270-2025-02

## Chassis complete, Article no: 70827002

Part no.	Art.no.	QTY.	Name	Drawing no.
1	70820008	2	Wheelcap back	8200-25.3
2	70820007	2	Wheelcap front	8200-26.3
3	70820009	1	Tube cap	8200-27
4	70820012	2	Bearing bush POM Ø15.1x11.1x15 mm	8200-29
5	80820024	2	Castor Ø75 unbraked double roller	Manner-Ø75-Zonder rem
6	80820023	2	Castor Ø100 braked double roller	Manner-Ø100-Met rem-laag
7	N.A. (Set with chassis complete)	1	Chassis	8270-11
8	N.A. (Set with chassis complete)	1	Leg set (L+R)	8200-13.1
9	N.A. (Set with chassis complete)	1	Transmitter	8270-16
10	N.A. (set with chassis)	2	Pull strip leg mechanism	8270-17
11	70827005	1	Pedal Lotus XL	Pedaal Lotus XL
12		1	Cover leg mechanism	8270-14
13		2	Bush Leg	8200-18
14		4	Washer	8200-19
15		1	Locking plate	8200-20
16		2	Screw M5x8 ISO7380 A2	M5x8 ISO7380
17		3	Screw M5x16 ISO7380 A2	M5x12 ISO7380
18	70820020	1	Screw M5x16 ISO7380 A2 Dry-loc+Lurinox	M5x16 ISO7380 Dry-loc+Lurinox
19		3	Locknut M5 DIN985	M5 DIN985
20		4	Screw M6x55 DIN7991 A2	M6x55 DIN7991
21		4	Locknut M6 DIN985 A2	M6 DIN985
22	70820021	2	Screw M8x16 ISO7380 A2 Dry-loc+Lurinox	M8x16 ISO7380 Dry-loc+Lurinox
23		4	Washer M8 DIN125 PA	M8 DIN 125 PA
24		2	Washer M8 Ø40	Carrosseriering M8x40
25		4	Flanged Slide bearing 8x11x6	08x11x07
26		2	Screw M10x70 DIN912 A2	M10x70 DIN912
27		2	Screw M10x70 DIN7991 A2	M10x70 DIN7991
28		4	Locknut M10 DIN985 A2	M10 DIN985
29		4	Washer M10	M10 DIN 125
30		4	Ball bearing	radial ball bearing_68_din
31		2	Ball bearing	radial ball bearing_68_din
32		4	Quick lock for axles Ø8	Snelborger 8mm 51445.080.001

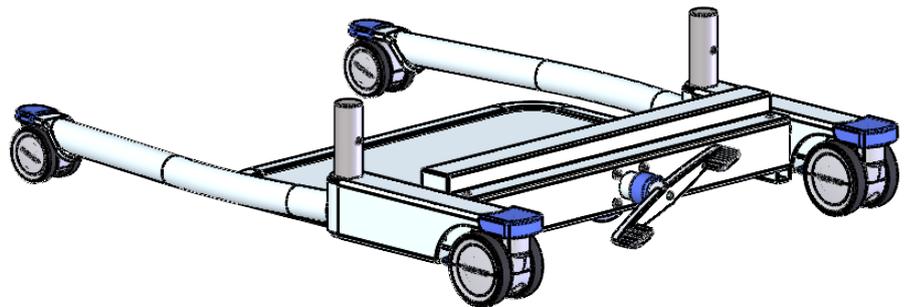

Drawn by:	KA	Date	19-10-2023
Revised	1 /KA	Date	31-07-2024
Revised	2 /KA	Date	15-01-2026

## Overview parts Chassis complete Lotus XL

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Draw.no. EXPV8270-2025-02