

From serial no. 8250.26.001

Drawn by: KA	Date: 19-10-2023
Revised: 1 /KA	Date: 26-07-2024
Revised: 2 /KA	Date: 05-02-2026

Overview parts Lotus compact

Copyright reserved by law

Unit of measurement: mm

Lopital Nederland bv

Draw. no.: EXPV8250-2025-02

Lotus compact, Article no: 5900 8250

Part no.	Art.no.	QTY.	Name	Drawing no.
1	70820001	1	Chassis complete	8200-Chassis complete
2	70825001	1	Seat frame compact (set)	8250-12.1
3	70820003	1	Frame handgrip	8200-14.1
4	70820014	4	Leaf spring	8200-36
5	70820004	2	Seat shell	8200-21.5
6	70820005	2	Seat cap	8200-22
7	70820013	2	Cap	8200-23.1
8	70820006	1	Foot Plate	8200-24
9	70820009	2	Tube cap	8200-27
10	70820010	1	Knee support	8200-30.2
11	70820019	4	Cross Knob PA 40 mm M6 grey	Kruisknop M6
12	70560049	2	Handgrip	5600-33.1
13	70820031	2	Slide bearing 8x15x10	8200-39
14		4	Screw M4x12 DIN7985 A2	M4x8
15		2	Screw M6x10 DIN965 A2	M6x10 DIN965
16	70820021	4	Screw M8x16 ISO7380 A2 Dry-loc+Lurinox	M8x16 ISO7380 Dry-loc+Lurinox
17		2	Screw M8x20 A2 ISO7380 flange	M8x20 ISO7380 Kraag
18		4	Nut cap M6	Moerkap M6
19		2	Countersunk cross slotted screw M6x20	M6x20 DIN965 A
20		2	Countersunk cross slotted screw M6x35	M6x35 DIN965 A



Drawn by: KA	Date: 19-10-2023
Revised: 1 /KA	Date: 26-07-2024
Revised: 2 /KA	Date: 05-02-2026


Overview parts Lotus compact

Copyright reserved
by law

Unit of measure-
ment: mm

Lopital Nederland bv

Draw. no.: EXPV8250-2025-02

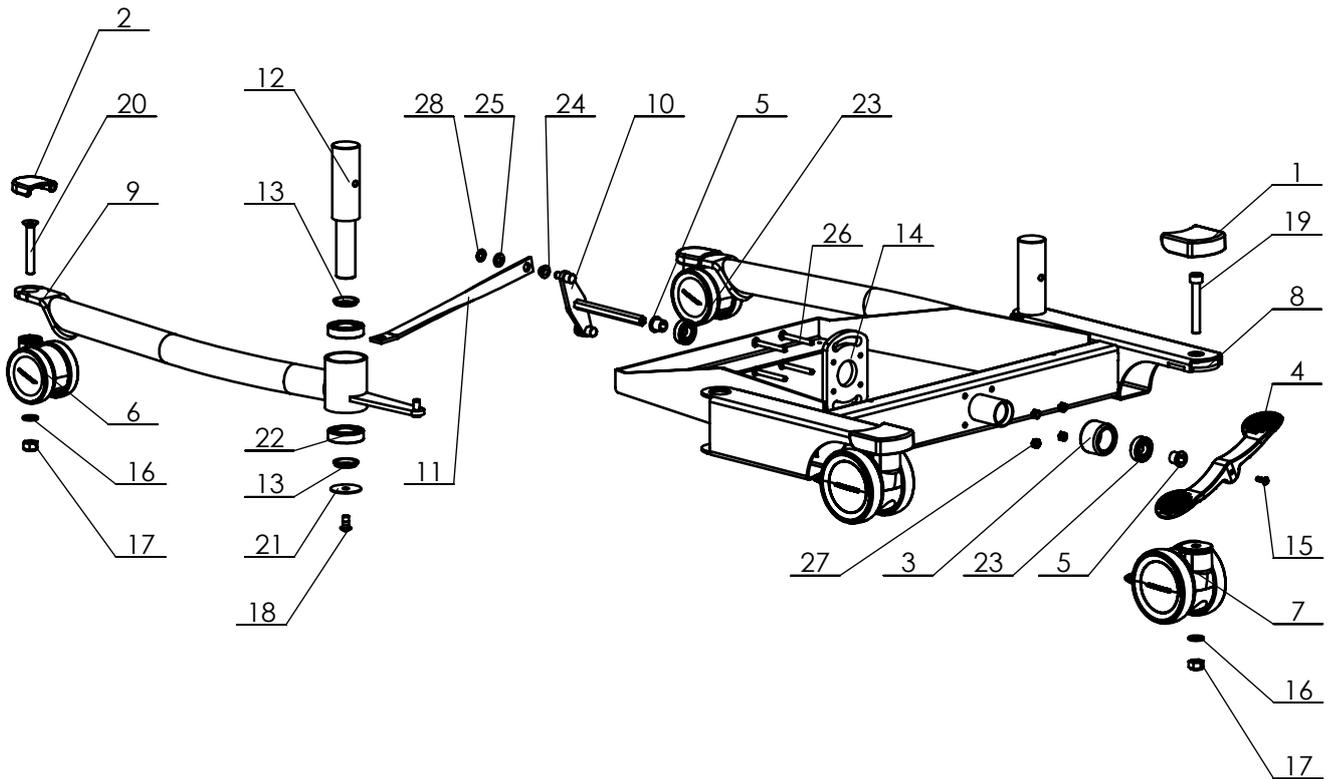

P:\SW-Ftek\Lotus 8200\

Drawn by: KA	Date: 19-10-2023
Revised: 1 /KA	Date: 26-07-2024
Revised: 2 /KA	Date: 05-02-2026

Overview parts Chassis complete Lotus

Copyright reserved by law


Unit of measurement: mm

Lopital Nederland bv

Draw. no.: EXPV8250-2025-02

Chassis complete, Article no: 70820001

Part no.	Art.no.	QTY.	Name	Drawing no.
1	70820008	2	Wheelcap back	8200-25.3
2	70820007	2	Wheelcap front	8200-26.3
3	70820009	1	Tube cap	8200-27
4	70820011	1	Pedal Lotus	2000-32
5	70820012	2	Bearing bush POM Ø15.1x11.1x15 mm	8200-29
6	80820024	2	Castor Ø75 unbraked double roller	Manner-Ø75-Zonder rem
7	80820023	2	Castor Ø100 braked double roller	Manner-Ø100-Met rem-laag
8	N.A. (Set with chassis complete)	1	Chassis	8200-11.3
9	N.A. (Set with chassis complete)	1	Leg set (L+R)	8200-13.1
10	N.A. (Set with chassis complete)	1	Transmitter	8200-15.1
11	N.A. (set with chassis)	2	Plate	8200-16
12		2	Bush Leg	8200-18
13		4	Washer	8200-19
14		1	Locking plate	8200-20
15	70820020	1	Screw M5x16 ISO7380 A2 Dry-loc+Lurinox	M5x16 ISO7380 Dry-loc+Lurinox
16		4	Washer M10	M10 DIN 125
17		4	Locknut M10 DIN985 A2	M10 DIN985
18	70820021	2	Screw M8x16 ISO7380 A2 Dry-loc+Lurinox	M8x16 ISO7380 Dry-loc+Lurinox
19		2	Screw M10x70 DIN912 A2	M10x70 DIN912
20		2	Screw M10x70 DIN7991 A2	M10x70 DIN7991
21		2	Washer M8 Ø40	Carrosseriering M8x40
22		4	Ball bearing	radial ball bearing_68_din
23		2	Ball bearing	radial ball bearing_68_din
24		4	Flanged Slide bearing 8x11x6	08x11x07
25		4	Washer M8 DIN125 PA	M8 DIN 125 PA
26		4	Screw M6x55 DIN7991 A2	M6x55 DIN7991
27		4	Locknut M6 DIN985 A2	M6 DIN985
28		4	Quick lock for axles Ø8	Snelborger 8mm 51445.080.001

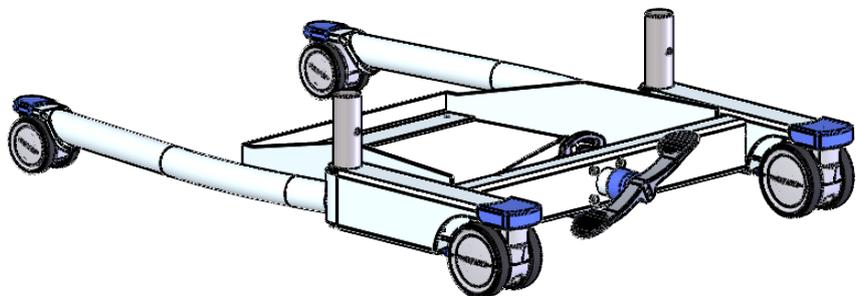

Signed	KA	Date	19-10-2023
Revised	1 /KA	Date	26-07-2024
Revised	2 /KA	Date	05-02-2026

Overview parts Chassis complete Lotus

Copyright reserved
by law

Unit of measure-
ment: mm

Lopital Nederland bv

Draw.no. EXPV8250-2025-02